



Electric Lifting Table User Manual GWN-142




Instruction for Electric Lift Grooming Table N-142

Parts List




Plastic cap is for protection only


1. Height regulating lever
2. Lifting switch
3. Regulating screw
4. Switch for the table top light
5. Table top light
6. Glass table top
7. Electric lifting pump
8. Wheel
9. Strap
10. Traversing lever
11. Fix screw for the arm
12. Grooming table arm

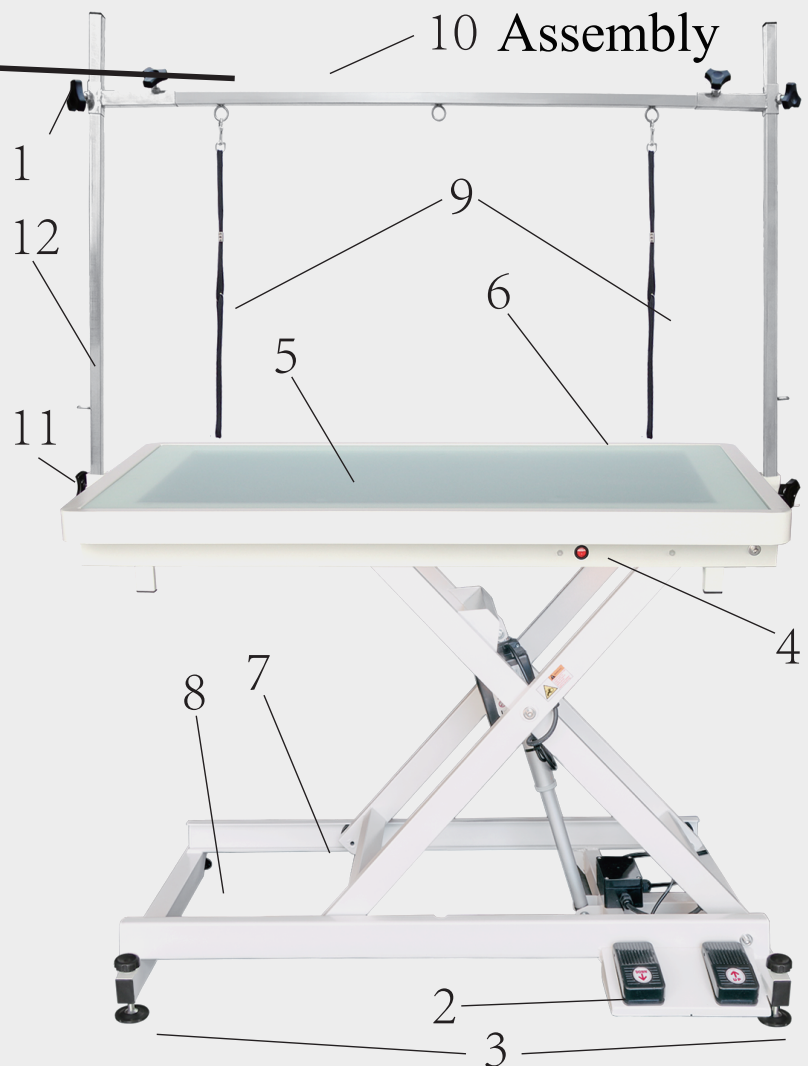

Assembly of The Grooming Arm

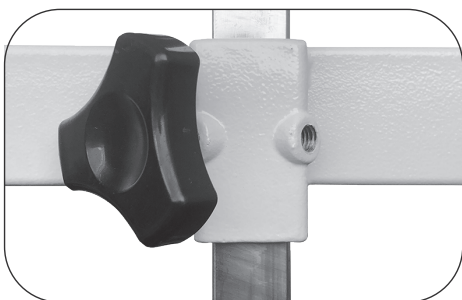

1. Put the grooming arm(12) into the hole and tighten up the both sides' screws(11) to fix the grooming arm.

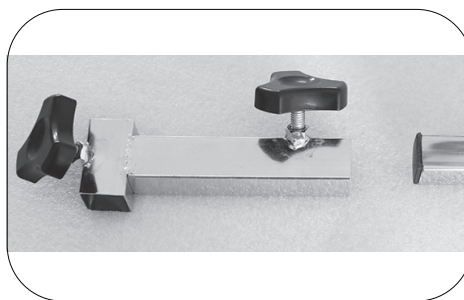


2. Put the traversing lever(10) into the height regulating lever(13) and fix the screw.




3. Regulate the height for the arm according to the users' requirements and then fix the regulating screws on the height regulating lever (13).

Instruction Manual:

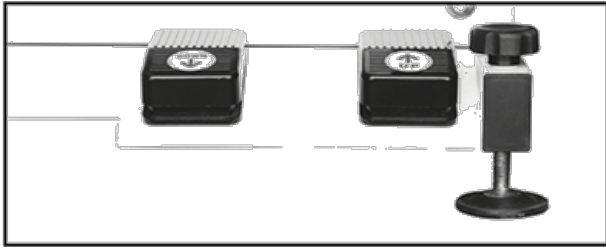

1. Loosen the regulating screws to adjust the

2. Tighten the regulating screws to ensure the grooming table is stable.

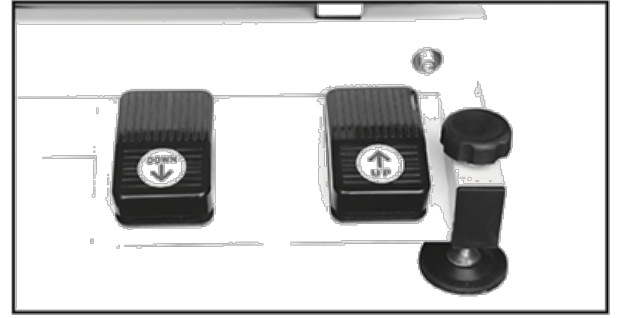

3. Plug the cord into a standard 110V/230V wall outlet. Press the 'Up' or 'Down' button on the lifting switch as required.

4. Choose turn on the switch for the table top light(4) or not according to the operation requirement.

⚠ WARNING

Important Safety Information

The operator must read and fully understand this instruction manual before using the appliance. Failure to follow the operating and safety instructions may result in serious injury or death.

Operating Procedure

- Check the product for any damage before use.
- Switch on the power supply and lower the table to a suitable height for safely loading the dog.
- Carefully place the dog onto the table and adjust the height as required for comfortable working conditions.
- Ensure the dog is securely positioned and, where appropriate, restrained before operation.
- Ensure the surrounding area and floor are clean and dry before making any height adjustments.
- Switch on the power supply and adjust the table to the required working height.
- Avoid unnecessary or continuous operation of the lifting mechanism.
- Ensure the dog remains centrally positioned on the table during operation.
- Once grooming is complete, lower the table to a safe height for removal.
- Switch off the power supply once the desired height is reached.
- Carefully remove the dog from the table.

Safety & Operating Instructions

- This appliance is intended solely for professional dog grooming. It must not be used for lifting people or for any other purpose.
- This appliance may only be operated by trained professionals aged 18 years or over.
- Children must not operate or play with the equipment.
- Do not exceed the maximum load capacity of 100 kg. Ensure the dog is positioned centrally at all times.
- Ensure the unit is installed on a stable, level surface and that all components are correctly assembled prior to use.
- Inspect all structural components, including welding points on the lifting frame, before each use to ensure there are no signs of damage, cracking, or fatigue; do not operate the appliance if any defects are identified.
- Inspect painted surfaces regularly; any damaged or chipped paintwork must be restored immediately to prevent corrosion and deterioration of structural components.
- Always ensure the surrounding area and floor are clean and dry before use to reduce the risk of slips, trips, or electrical hazards.
- Ensure all electrical connections are protected by an IP66-rated socket & suitable waterproof cover when operating.
- Electrical components must be protected from water ingress at all times. Do not allow water to come into contact with plugs, sockets, controls, or cables.
- Do not tamper with, interfere with, or attempt to access internal electrical components. Do not insert, poke, or prod objects into any openings, controls, or electrical parts of the appliance.
- Keep hands, feet, and other body parts clear of moving parts, particularly the lifting mechanism, as this may result in crushing or pinching injuries and could result in serious injury or death.
- Never leave a dog unattended on the table and maintain supervision at all times during operation.
- Do not use the equipment if the power cable, plug, transformer, actuator, or foot pedal is damaged.
- Do not connect the appliance to an overloaded circuit or shared outlet where this may present a risk.
- Ensure all cables are positioned safely to prevent damage, strain, tripping hazards, or access by dogs.
- Do not operate the lifting mechanism continuously for more than 2 minutes. Allow a minimum rest period of 18 minutes between cycles to prevent overheating.
- The expected service life of this appliance is 3 years under normal use and maintenance.
- Always disconnect the appliance from the power supply when not in use and at the end of each working day.
- Disconnect from the power supply before cleaning, inspection, or maintenance.
- Clean using a dry or slightly damp cloth only. Do not use corrosive, abrasive, or solvent-based cleaning agents.
- Do not use water to clean electrical components, including the controller, foot pedal, or actuator, due to risk of electric shock.
- In the event of abnormal operation, stop use immediately, isolate the appliance from the power supply, and do not attempt to repair or disassemble the unit. Contact your supplier for assistance.
- Do not modify the equipment in any way. Use only approved parts and accessories.

- Ensure the working environment is well-lit, clean, and free from obstacles; always ensure the dog is correctly positioned on the table and not beneath or near moving parts during operation.
- Always use appropriate restraints where necessary to ensure the safety of the dog.
- Regularly inspect the appliance to ensure it remains in safe working condition. All repairs must be approved by Groomers World.
- Ensure all safety labels remain present, clean, and clearly legible at all times; if not, contact us at info@groomersworld.co.uk.
- This manual may have expired. For product updates and the latest instruction manual, visit: www.groomersworld.co.uk

For all warranty information, please contact us at info@groomersworld.co.uk.

Warranty Exclusions

This warranty does not cover faults or damage caused by misuse, neglect, accidental damage, failure to follow instructions, unauthorised repairs or modifications, normal wear and tear, unsuitable operating conditions, improper maintenance, transportation damage after delivery, or use of non-approved parts or accessories. This warranty does not affect your statutory rights under UK consumer law.

Limitation of Liability

To the fullest extent permitted by law, the supplier shall not be liable for any indirect or consequential loss, including loss of profits, revenue, goodwill, or third-party claims. Nothing in this clause excludes or limits liability for death or personal injury caused by negligence, fraud or fraudulent misrepresentation, or any liability that cannot be excluded under UK law.

GW Equipment Ltd		Electric Lifting Table	
Name	Pet Grooming Table	Weight	42 kg
Model	GW N-142	Input Voltage/ Frequency	100-240V~50/60Hz
Size	116L*60W*27-102H cm	Output Watt	36W
Lifting Height	27-102cm	Output	24V=,Max1.5A
Static Load	90 kg	Manuf No	
Max Load	100 kg	Manuf Date	



WARNING
Read and fully understand operator's manual before using this machine.
Failure to follow operating instructions could result in death or serious injury.


Every 2 min lifting requires 18 min rest








GW Equipment, Unit 11 Sunflag Business Park, Durham, DH1 1TN | tel: 07511785417
info@groomersworld.co.uk
 MADE IN CHINA

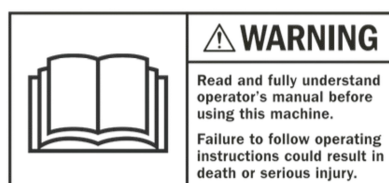

MADE IN CHINA